


## RA-MM Wall Support Kit & RA-MM Wall Support Kit RETRO

### Installation Instructions.

When fitting the Wall Strap Bracket Kit included in either of the above kits, please ensure both brackets are fixed around the UPPER part of the LOWEST section on both lifting legs, 20mm from the top edge as seen in the image below. The whole bracket assembly then needs to be fixed to the solid wall at the same height.

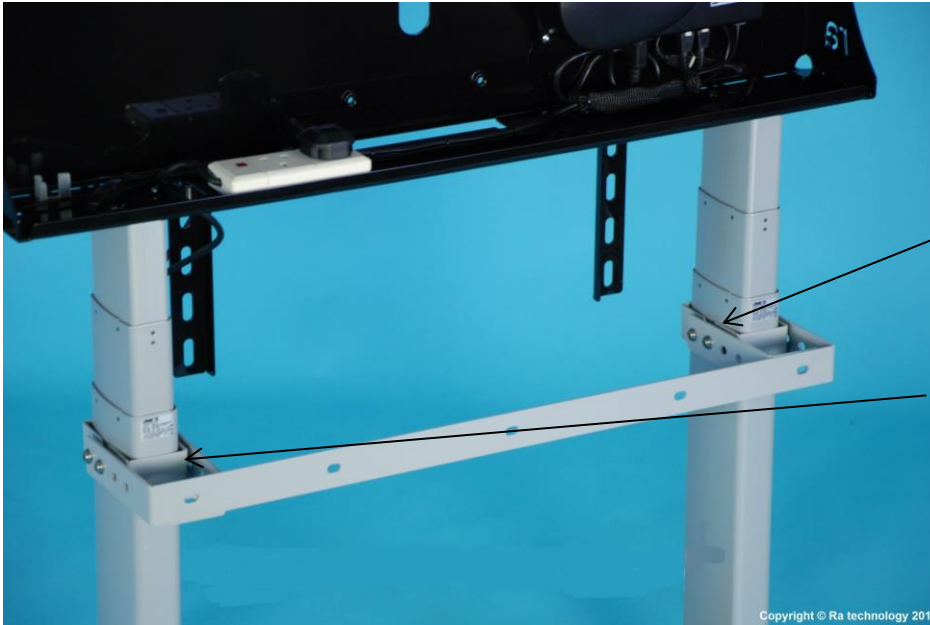

Wall Strap Brackets must be fitted to the upper part of the lowest sections of each lifting leg as shown in the image, and then fixed back to a solid wall

When installing the larger floor plate included in the **RA-MM Wall Support Kit RETRO** (if not already fitted), the existing floor plate fitted to the riser (with 7 fixing holes) needs to be replaced with the larger floor plate by removing the 4 x M6 countersunk bolts at the base of each leg & refitting with the larger floor plate in place.



RA-MM-Wall Support Kit.  
Deeper Floor Plate without any mounting holes

For any additional support with these kits please contact Ra technology Customer Services on 01484 475950